

# Dale® NasoGastric Tube Holder

## Superior Security for Naso-Gastric Tubes


The Dale NasoGastric Tube Holder is ideal for holding feeding and aspiration tubes such as Salem sump tubes, Levin stomach tubes, naso-enteric feeding tubes, and naso-gastric intestinal tubes. Skin-friendly adhesive stays in place for extended periods, yet is easy to remove.

### Application Instructions

1. After tube is in place, cleanse tube and nose with alcohol wipe.
2. Peel holder completely from liner, starting at nose pad. Apply nose pad to nose (Figure 1).
3. Choose tab closest to tube and begin to tightly wrap the tube in a spiraling fashion (Figure 2).
4. Wrap remaining tab using same spiraling method (Figure 2).

To remove: Simply use the non-adherent tips to unwrap tabs from tube.

**For single patient use.**

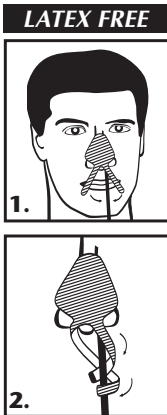

## Ordering Information

#H8410**1601** Dale® NasoGastric Tube Holder

Fits all. Individually packaged. Sold in boxes of 50.