

Dale[®] Bridle[™]

Anti-Disconnect Device for Tracheostomy

Product Description

Designed to secure the connection of the breathing circuit to the tracheostomy tube. The Dale Bridle secures closed tracheal suction systems to the tracheal tube. The Dale Bridle can also connect elbow, straight or swivel adapters to the breathing circuit.

Application Instructions

1. Position Dale Bridle so hole is placed over the closed tracheal suction device or connector (Fig. 1).
2. Wrap tab (A) around closed tracheal suction device or connector (Fig. 1). Press tab to secure.
3. Slip long Velcro[®] arms (B) up behind wings of trach plate & against patient's neck (Fig. 2). Press Velcro arms (B) to secure (Fig. 3). Check that all 3 tabs are secured.

Caution: The Dale Bridle does not replace the need for a disconnect alarm.

Use in supervised care settings only. Monitor to prevent unintentional disengagement. Do not wash.

For single patient use.

For maximum stabilization we recommend using with the Dale[®] Tracheostomy Tube Holder.

Ordering Information

#H8410**2481** Dale[®] Bridle[™]
Large. Individually packaged. Sold in boxes of 10.

#H8410**2491** Dale[®] Bridle[™]
Small. Individually packaged. Sold in boxes of 10.

Dale medical products are available from your preferred hospital/medical supply dealer. **Latex Free.**

US Patent Number 5,924,421

Velcro[®] is a registered trademark of Velcro USA, Inc.



DALE MEDICAL PRODUCTS, INC.

7 Cross Street, P.O. Box 1556, Plainville, MA 02762-0556 USA

TEL: 800-343-3980 INT'L: 508-695-9316 FAX: 508-695-6587

www.dalemed.com



EU Representative:

Medlink Europe B.V. Nieuwegein, NL.

Tel: 31.30.287.0458 FAX: 31.30.280.0388

