

# Dale<sup>®</sup> Nasal Dressing Holder

Product Number **H84106001**  
One Size Fits All

*Tape-Free Securement*



## Product Description

The Dale Nasal Dressing Holder replaces adhesive tape to hold nasal dressings in place following rhinoplasty, septoplasty, sinus surgery, facial trauma and nose bleeds. Totally adjustable, it eliminates skin irritation, simplifies dressing changes and improves patient comfort.



## Application Instructions

- Center plastic section of the holder over the nasal dressing and under the nose. Secure by placing ear loops around the patient's ears.
- To adjust size (ear to nose)
  - Lift end of strap and reposition ear loop assembly to the desired length. Repeat for other side.
  - Fold end of the strap over the ear loop assembly. Excess strap material can be removed with scissors.

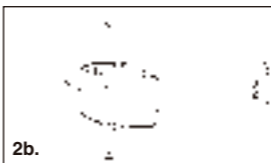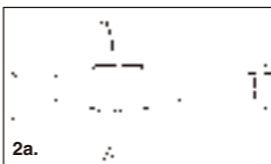

## To Change Dressing

To change nasal dressing, remove one ear loop, change dressing, and replace ear loop around the patient's ear.

**For single patient use.  
One size fits all.**

## Ordering Information

#H84106001 **Dale<sup>®</sup> Nasal Dressing Holder**  
Sold in boxes of 10.

Dale medical products are available from your preferred hospital/medical supply dealer. **Latex Free.**

US Patent Number 5,284,469; 5,362,303; Des. 354,812



DALE MEDICAL PRODUCTS, INC.

7 Cross Street, P.O. Box 1556, Plainville, MA 02762-0556 USA

TEL: 800-343-3980 INT'L: 508-695-9316 FAX: 508-695-6587 www.dalemed.com

**EU Representative:**  
**Medlink Europe B.V. Nieuwegein, NL.**

REV B

Tel: 31.30.287.0458 FAX: 31.30.280.0388