


Dale® Bendable ArmBoard

All-In-One System


Product Description

The Dale Bendable ArmBoard replaces traditional armboards. The Bendable ArmBoard allows the hand and wrist to maintain their natural position while securely and comfortably preventing catheter movement. The ArmBoard positions and secures the hand and wrist during insertion and monitoring of arterial lines in the operating room, recovery room, and patient care units. The ArmBoard can be custom-shaped to any desired position. It is covered in a soft, comfortable material, and fastened with adjustable VELCRO® straps. It can be used to secure the thumb during extension of the wrist for better access to the radial artery. This leads to more trouble-free A-Line insertions and facilitates invasive monitoring of arterial blood pressure and arterial blood gases for extended periods.

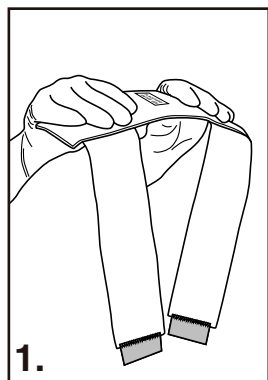

Application Instructions

1. Align wrist with ArmBoard and bend to desired position.
2. Wrap straps around hand and board to desired tension. Press VELCRO hook fasteners to secure.

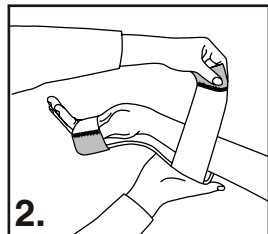

For single patient use.

Ordering Information

- #H8410**6501** Dale® Bendable ArmBoard
Large. Individually packaged. Sold in boxes of 10.
- #H8410**6511** Dale® Bendable ArmBoard
Medium. Individually packaged. Sold in boxes of 10.
- #H8410**6521** Dale® Bendable ArmBoard
Small. Individually packaged. Sold in boxes of 10.

Dimensions

- 9" x 4 3/4"
- 5 1/2" x 3"
- 4 1/2" x 1 3/4"

Dale medical products are available from your preferred hospital/medical supply dealer.

Latex Free.

US Patent Number 5,845,643

VELCRO® is a registered trademark of Velcro USA, Inc.



DALE MEDICAL PRODUCTS, INC.

7 Cross Street, P.O. Box 1556, Plainville, MA 02762-0556 USA
TEL: 800-343-3980 INT'L: 508-695-9316 FAX: 508-695-6587
www.dalemed.com

CE EU Representative:
Medlink Europe B.V. Nieuwegein, NL.
Tel: 31.30.287.0458 FAX: 31.30.280.0388